Description
<div> <div>①</div> <div>Cuff</div> </div> <div> <div>②</div> <div>Cuff Tube</div> </div> <div> <div>③</div> <div>Cuff Connector A</div> </div> <div> <div>④</div> <div>Cuff Connector B</div> </div>

Choosing the connector

• For blood pressure monitors that have a white, silver or dark coloured cuff socket on the device, use **Connector A** ③ with the grey connector pin.

• For blood pressure monitors that have a yellow coloured cuff socket on the device, use **Connector B** ④ with the yellow connector pin.
☞ To change the connector, pull the connector out of the cuff tube and insert the required connector into the cuff tube as far as it will go.

Cleaning the cuff

Carefully remove spots on the cuff with a damp cloth and soapsuds.

☞ The bladder must lay straight in the cuff cover, not folded.

☞ Do not use fabric softener.

⚠ WARNING: Under no circumstances may you wash the inner bladder!

⚠ The cuffs are sensitive and must be handled with care.

Guarantee

The cuff has a functional guarantee (bladder tightness) for 2 years.
• Not included are sleeve and cuff enclosure due to wear.
• The guarantee is only valid on presentation of the purchase receipt.
• The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

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